

I. PRODUCT DESCRIPTION

PRODUCT: OPTI-TEARS™ Soothing Drops

PRODUCT DESCRIPTION

DESCRIPTION: OPTI-TEARS® Soothing Drops is a sterile, isotonic, aqueous solution that contains Dextran, sodium chloride, potassium chloride and hydroxypropyl methylcellulose with edetate disodium 0.1% and POLYQUAD® (polyquaternium-1) 0.001% as preservatives.

ACTIONS: OPTI-TEARS® Soothing Drops moisten your lenses, and help to remove particulate material that may cause irritation and/or discomfort.

INDICATIONS (Uses): OPTI-TEARS® Soothing Drops may be used with daily and extended wear soft (hydrophilic) contact lenses for the following:

- Moistening of daily wear lenses while in the eyes during the day.
- Moistening of extended wear lenses upon awakening and as needed during the day.
- Moistening of extended wear lenses prior to retiring at night.

Placing 1 or 2 drops of OPTI-TEARS® Soothing Drops on the eye followed by blinking 2 or 3 times will relieve minor irritation, discomfort or blurring which may occur while wearing lenses.

CONTRAINDICATIONS (Reasons Not To Use): Patients allergic to any ingredient in OPTI-TEARS® Soothing Drops should not use this product.

WARNINGS: Never touch the dropper tip of the container to any surface, since this may contaminate the solution.

- | |
|---|
| PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN SERIOUS INJURY TO THE EYE. It is essential that you follow your eye care practitioner's direction and all labeling instructions for proper use of your lenses and lens care products. EYE PROBLEMS, INCLUDING CORNEAL ULCERS, CAN DEVELOP RAPIDLY AND LEAD TO LOSS OF VISION; THEREFORE, IF YOU EXPERIENCE EYE DISCOMFORT, EXCESSIVE TEARING, VISION CHANGES OR REDNESS OF THE EYE, IMMEDIATELY REMOVE YOUR LENSES AND PROMPTLY CONTACT YOUR EYE CARE PRACTITIONER. |
|---|
- All contact lens wearers must see their eye care practitioner as directed. If your lenses are for extended wear your eye care practitioner may prescribe more frequent visits.
- Keep this and all medications out of the reach of children.

PRECAUTIONS:

- Always wash and rinse your hands before handling your lenses.
- Store at room temperature.

Reference: 341940 (7/88)

ND09MF.BH

PRODUCT: OPTI-TEARS™ Soothing Drops

- Keep container tightly closed when not in use.
- Use before the expiration date marked on the containers and cartons.

ADVERSE REACTIONS (Problems and What To Do):

The following problems may occur while wearing contact lenses.

- Eyes stinging, burning or itching (irritation).
- Excessive watering (tearing) of the eyes.
- Unusual eye secretions.
- Redness of the eyes.
- Reduced sharpness of vision (visual acuity).
- Blurred vision.
- Sensitivity to light (photophobia).
- Dry eyes.

If you notice any of the above problems, immediately remove and examine your lenses.

If the problem stops and the lenses appear to be undamaged, thoroughly clean, rinse and disinfect the lenses and reinsert them. If the problem continues or a lens appears to be damaged, IMMEDIATELY remove your lenses and IMMEDIATELY consult your eye care practitioner. Do not reinsert a damaged lens.

If any of the above symptoms occur, a serious condition such as infection, corneal ulcer, neovascularization or iritis may be present. Seek immediate professional identification of the problem and treatment to avoid serious eye damage. See your Instructions for Wearers booklet for more information.

DIRECTIONS: OPTI-TEARS® Soothing Drops may be used as needed throughout the day. If minor irritation, discomfort, or blurring occur while wearing lenses, place 1 or 2 drops on the eye and blink 2 or 3 times. If discomfort persists, immediately remove lenses and immediately see your eye care practitioner.

HOW SUPPLIED: OPTI-TEARS® Soothing Drops are supplied in sterile 5 and 15 ml plastic bottles. The containers and cartons are marked with a lot number and expiration date.

LENSES: OPTI-TEARS® Soothing Drops are indicated for use with daily and extended wear soft (hydrophilic) contact lenses as recommended by your eye care practitioner.

Reference: 341940 (7/88)

ND09MF.BH